



B.15 Quick Guide #15: Temporarily Removing the D-Scribe CD

(Refer to CAREpoint Users Manual Section 7 for more information)



- Using the on-screen Keypad , press the sequence: **#UNLOCKCD** (**#-8-6-5-6-2-5-2-3**).
 - After a few moments, the CD drawer will open. If it does not open automatically, Press the eject button on the CD-ROM.
- Remove the D-Scribe CD, and place in a safe place. This CD needs to be reinserted once the CD drive is no longer being used.
- Insert the CD that you will need to temporarily use into the CD drawer.
- Close the CD drawer.
- Perform any operations that will utilize the CD that was just inserted. (i.e. Install Printer drivers, load documents into InfoManager)
- Once these operations are completed, press the eject button on the CD-ROM and remove the CD from the CD drawer.
- Reinsert the D-Scribe CD and close the CD drawer.



- Using the on-screen Keypad , press the sequence: **#LOCKCD** (**#-5-6-2-5-2-3**).
 - The CD drawer will now be locked by the Log Recorder software.
- Return to Normal Operations.

End