


QG0001A - Applying Rosetta-DS Auto Updates


Note: Must be logged in to Windows as an Administrator to perform updates.


- 1) When Rosetta-DS starts a WyUpdate window will appear. This window will provide some information about the update.

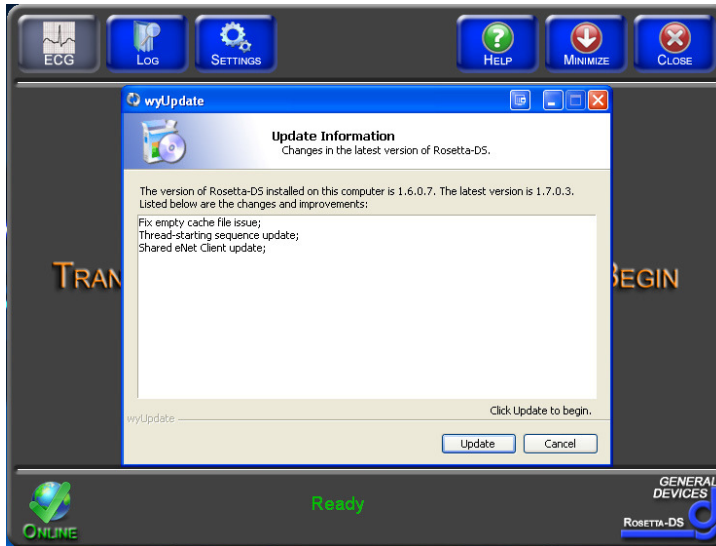



Click the  button to start the update process.


- 2) A “Downloading & Installing Updates” window will appear.

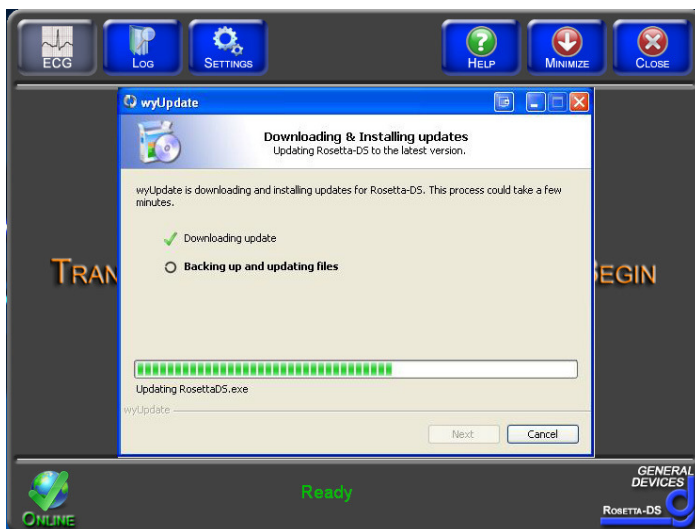

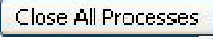- 3) During the installation process, a “Close processes” window may appear. It will automatically try to close the RosettaDS.exe. If it can not automatically close the running process, click the  button.

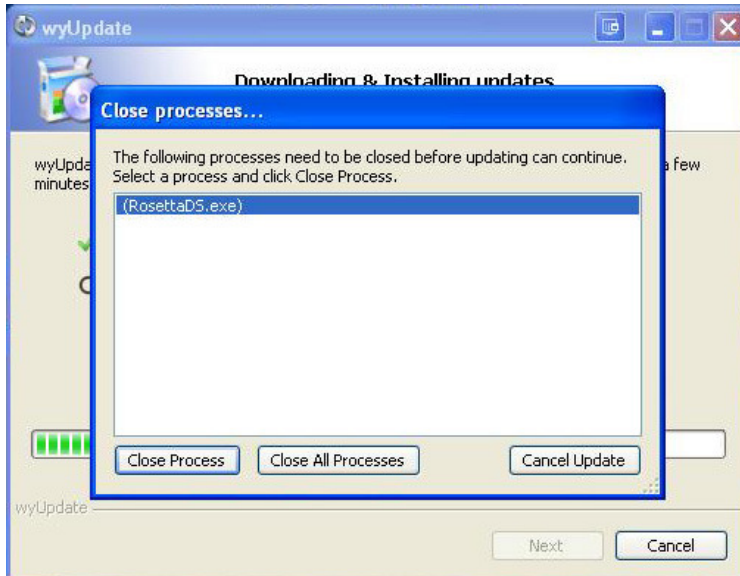

- 4) It will proceed with the update. Once the update is complete, click the  button. To restart Rosetta-DS, double click the Rosetta-DS shortcut on the Desktop.

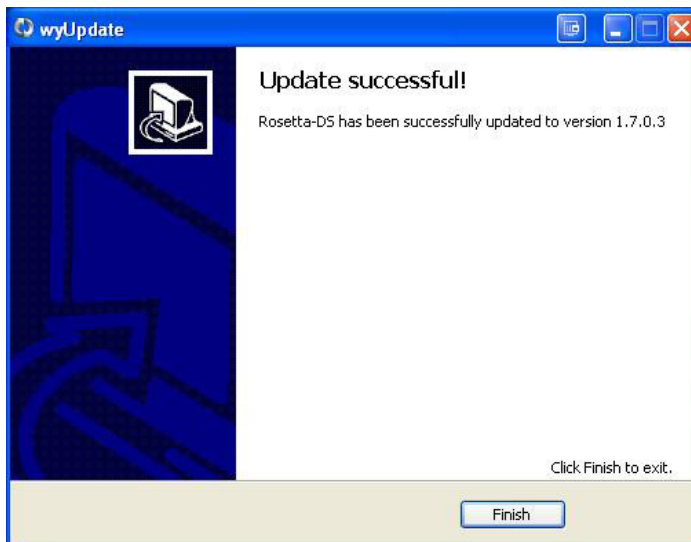

-END PROCEDURE-