

General Devices Rosetta System Compatibility Information:

Note: A signed copy is REQUIRED for acceptance of a purchase order.

Monitor/Defibrillators

The following is a list of compatible monitors / features for use with Rosetta-Lt.

- Single Lead ECG:
 - Any monitor/defibrillator with standard 1-Volt/mVolt ECG output
- 12-Lead ECG/Summary Data:
 - Medtronic/Physio Control LIFEPAK 12 and LIFEPAK 15
 - ZOLL M Series with “RS-232” serial/ECG port & Interlink/Data Relay, and Rev 38.5 or later software
Note: M-CCT model is NOT compatible.
 - ZOLL E Series with w/ Rev 1.01 or later software, and Interlink/Data Relay
 - Philips HeartStart MRx – with Rev 9.00 or later and B11 Rosetta option

Communications Devices (Radios/Phones)

Compatible:

- Landline Telephone: - Compatible (acoustic coupling only)
 - Any landline (wired) telephone can be used via acoustic coupling
- Cellular Telephone:
 - Analog cellular phone – Compatible (these networks are however discontinued)
 - iDEN * – Nextel (dial up, not PTT) *Sprint CDMA network is not compatible*
 - GSM * - (ATT/Cingular, T-Mobile)
* Compatible however, not all text/summary data may be received. Digital phones MUST be tested on the system to verify compatibility.
 - 2.5mm “Headset Jack” or connector adapter required for cable connection
- 2-Way Radio (portable or mobile)
 - Conventional Radio – VHF/UHF (Analog – i.e. HEAR, MED Ch, etc)
 - Trunked Radio – 800MHz with Analog “voice” channels

Note: Systems with poor RF coverage may not provide reliable operation. Be sure to test all equipment prior to use to assess performance.

NOT Compatible:

- Digital Radio – (digital voice channels) includes some newer 800 trunk systems (i.e. P25)
- CDMA (Verizon, Sprint, AllTel) digital cellular phone – NOT compatible

Contact General Devices’ Technical Support department regarding compatibility questions at:

(201) 313-7075 or support@general-devices.com

I have read and understand the above compatibility information:

Name: _____ Title: _____

Signature: _____ Date: _____

A signed copy is REQUIRED for acceptance of a purchase order. Please fax or email a completed and signed copy to General Devices.

**GENERAL
DEVICES**

1000 River Street, Ridgefield, NJ 07657
tel 201.313.7075 fax 201.313.5671
www.general-devices.com
sales@general-devices.com

