



## B.9 Quick Guide #9: Shutting Down the CAREpoint

(Refer to CAREpoint Users Manual Section 3 for more information)

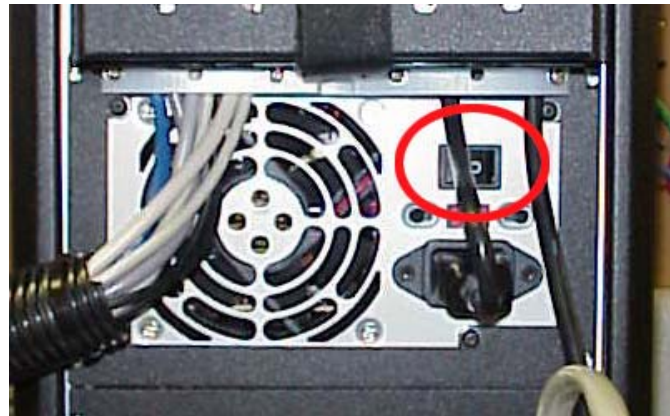


1. Using the on-screen Keypad, press the sequence; **#SHUTDOWN (#-7-4-8-8-3-6-9-6)**.
2. A message similar to "It is now safe to turn off the power" will appear
3. At this point, turn off the Power Supply on the back of the CAREpoint OPE (The black box containing the CD-ROM).
  - a. The switch for the Power Supply is located near the power cord plugged into the back of the CAREpoint OPE.

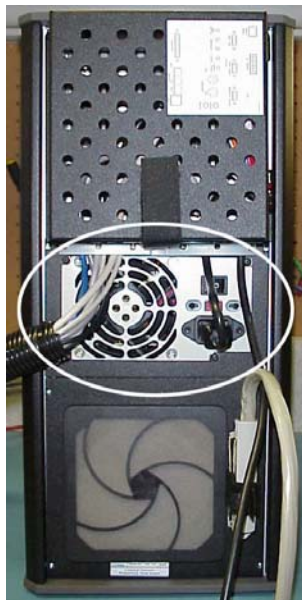


- b. Turn the Power Supply from the "ON" ( 1 ) position into the "OFF" ( 0 ) position.

Power Supply is ON when this side is pressed in.

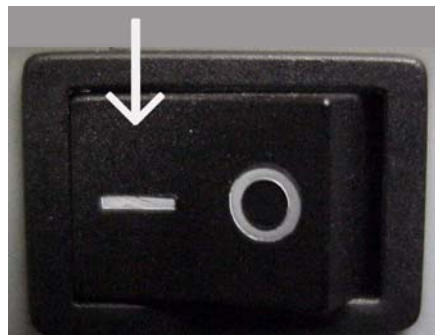


Power Supply is OFF when this side is pressed in.

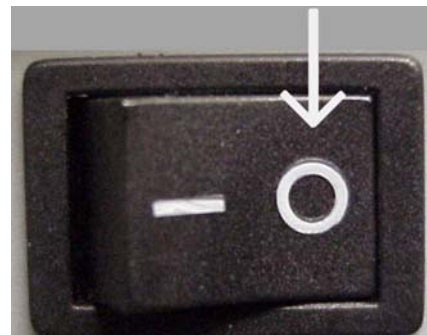


4. Leave the Power Supply off for approximately 10 seconds.
5. To power the CAREpoint on again, turn the Power Supply from the “OFF” ( 0 ) position into the “ON” ( 1 ) position.

Power Supply is OFF when this side is pressed in.

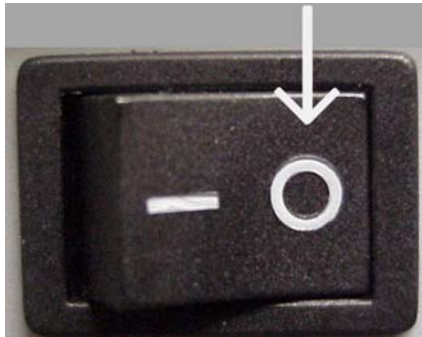

Power Supply is ON when this side is pressed in.

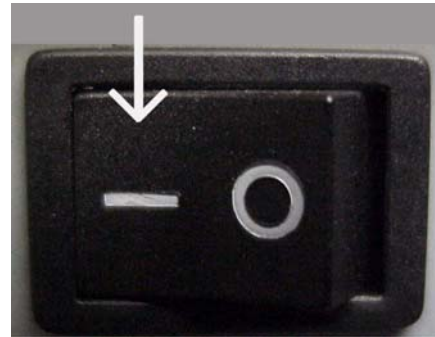

6. The CAREpoint will begin to reboot. The “Startup” window will appear in approximately one minute

End